

The activity you want.
The coverage you need.



Hour (GMT)	GBq	Millions of spheres per GBq
8:00	9,963	4.5
10:00	9,750	4.6
12:00	9,542	4.7
14:00	9,337	4.8
16:00	9,138	4.9
18:00	8,942	5.0
20:00	8,751	5.1



Hour (GMT)	GBq	Millions of spheres per GBq
8:00	7,686	5.8
10:00	7,521	5.9
12:00	7,361	6.0
14:00	7,203	6.2
16:00	7,049	6.3
18:00	6,898	6.4
20:00	6,751	6.6



Hour (GMT)	GBq	Millions of spheres per GBq
8:00	5,929	7.5
10:00	5,802	7.7
12:00	5,678	7.8
14:00	5,557	8.0
16:00	5,438	8.2
18:00	5,321	8.4
20:00	5,208	8.5



Hour (GMT)	GBq	Millions of spheres per GBq
8:00	4,574	9.7
10:00	4,476	9.9
12:00	4,380	10.2
14:00	4,286	10.4
16:00	4,195	10.6
18:00	4,105	10.8
20:00	4,017	11.1



Hour (GMT)	GBq	Millions of spheres per GBq
8:00	3,528	12.6
10:00	3,453	12.9
12:00	3,379	13.2
14:00	3,307	13.5
16:00	3,236	13.7
18:00	3,167	14.0
20:00	3,099	14.4

To calculate the number of spheres extracted from the vial: divide the extracted activity by the total activity of the vial, then multiply this number by 44.5 million.

Optimize SIRT with The SIR-Spheres® Advantage



ORDER-MAP-TREAT
Programme



FLEXdose Delivery Programme



INSY-90
The power of knowing
with SurePlan LiverY90

Intended Use: SIR-Spheres Y-90 resin microspheres are intended for implantation into hepatic tumors via the hepatic artery.
Indications For Use: SIR-Spheres Y-90 resin microspheres are indicated for the treatment of unresectable hepatocellular carcinoma (HCC) or unresectable metastatic liver tumors from primary colorectal cancer in patients refractory to or intolerant of chemotherapy.
Consult the Instructions for Use for a complete listing of indications, contraindications, side effects, warnings, and precautions.

Manufacturer
Sirtex Medical Pty Ltd
Shop 6, 207 Pacific Highway
St Leonards, NSW 2065
Australia
Tel: +61 2 9964 8400
Fax: +61 2 9964 8410

www.sirtex.com

@SirtexMedical

company/sirtex-medical-limited

Europe, Middle East & Africa
Sirtex Medical Europe GmbH
Joseph-Schumpeter-Allee 33
53227 Bonn
Germany
Tel: +49 228 1840 730


SurePlan™ is a trademark of MIM Software Inc. MIM Software Inc. is the legal manufacturer of the SurePlan LiverY90 Software.
SIR-Spheres® is a registered trademark of Sirtex SIR-Spheres Pty Ltd.
©2021 Sirtex Medical Europe GmbH

APM-EMEA-012-07-20 V4

SIRTeX

SIRTeX | SIR-Spheres®
Y-90 resin microspheres


FLEXdose Delivery Programme

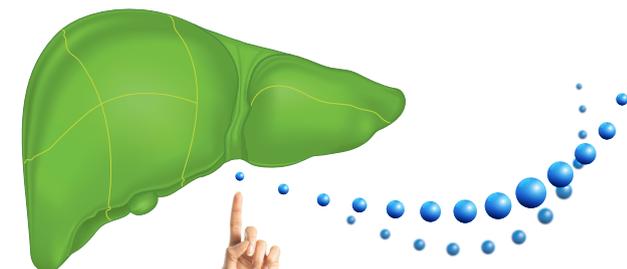

Advancing to the next level.

Optimize activity

Maximize coverage

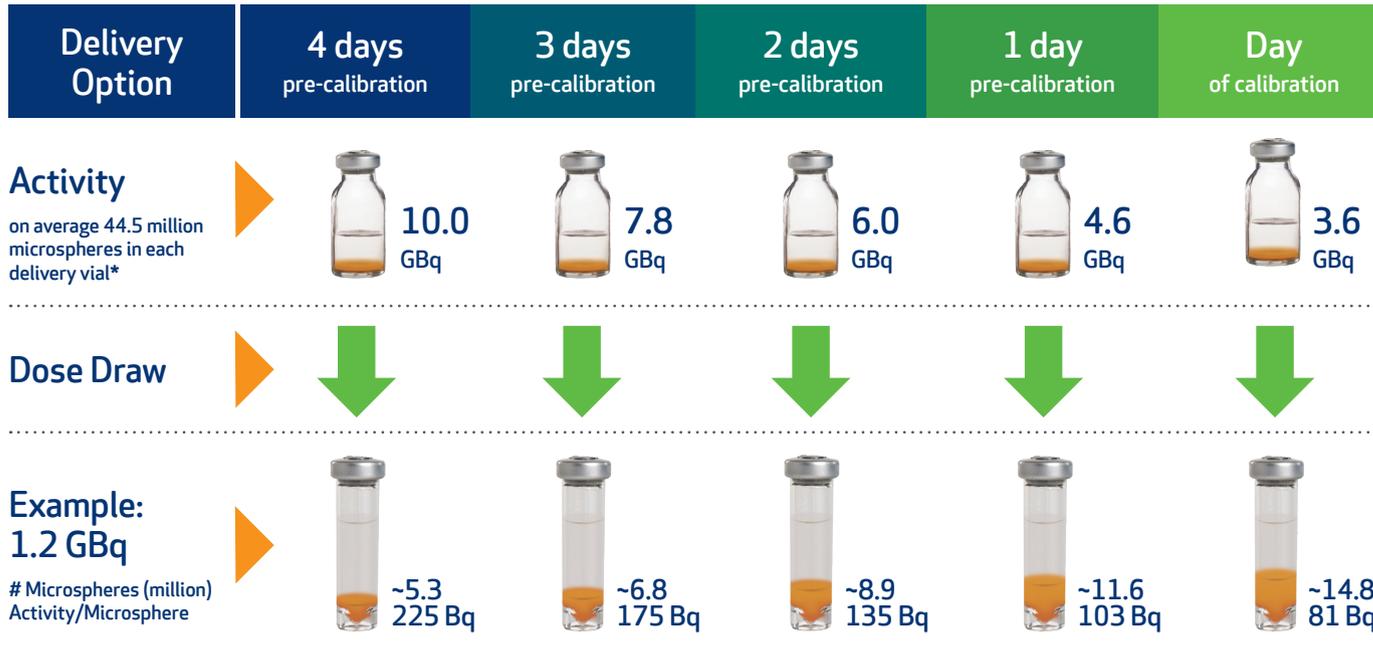


*Standard deviation of 2.6 million. Engineering test results on file. Activities for the morning of the delivery day at 7:00 am GMT.

Adapt your treatment


- for targeted liver volume
subsegmental, segmental, lobar
- for tumor vascularization
relatively hypovascular, hypervascular,
impaired arteries from previous treatments
- for treatment goal
enable surgery, palliative

The **ACTIVITY** you want
for liver-sparing
segmental approach

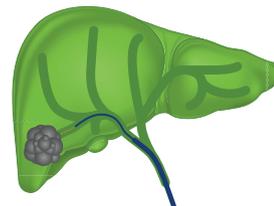

Microcatheter

The **COVERAGE** you need
in 1 vial to treat large
and/or multiple tumors

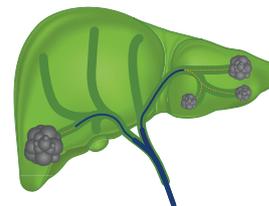

Microcatheter

The unique flexibility of SIR-Spheres Y-90 resin microspheres

Flexible treatment strategy

- Tailored activity for patient-specific needs
- Multiple injection sites
- Procedural approach can be adjusted on day of treatment
- Full control through angiographic visibility

Flexible treatment planning

- Multiple delivery options for an expedited patient path
- ORDER-MAP-TREAT programme, map and treat on the same-day or the same-stay

Flexible dose draw

- Activity ready to use when required
- Possibility to adapt the activity until the very last minute